



Safety Gate: Rapid Alert System for dangerous non-food products

Alert number	INFO/00080/20
Category	Protective equipment
Type of alert	
Product user	Consumer
Product	Particle filter mask
Brand	
Name	Non-medical protective mask
Type / number of model	
Barcode	
Batch number	Production year: in 2020
OECD Portal Category	85000000 - Safety / Protection - DIY
Description	White respiratory protective half-face mask of category KN95.
Packaging description	The product is sold in a plastic bag containing 3 pieces.
Country of origin	People's Republic of China
Alert submitted by	Hungary
Counterfeit *	UNKNOWN
Risk type	Health risk / other
Technical defect	The product advertises a protective potential against particles and microorganisms but its filtering capacity has not been certified by a relevant European body. Consequently, the product might not fulfil the health and safety requirements and properly protect when combined with additional measures. / The product does not comply with the Personal Protective Equipment Regulation.
Risk	
Measures adopted by notifying country	Import rejected at border

Images



WEARING METHOD

1



Hold the mask with your fingertips in the nose clip position. Open the mask slowly.

2



Place the nose clip above the mask, put the mask on the face, and press the nose clip at the same time to ensure the tightness of the mask.

3



Hang the ear strap on both ears and adjust the length of the earband at the same time for more comfortable wearing.

4



Check: Cover your mouth and breathe quickly. Feel the air leak from the nose. Please tighten the nose clip. If you feel the air leaks from the edge, please readjust the ear loop until the mask fits tightly the face.

ATTENTION

1. Non-medical protective mask adopts three-dimensional cutting to maintain breathable space and can be adjusted by using Nose bridge for best fit to face shape.
2. Wearing method: It is recommended to replace it every 4 hours. No washing and reuse it.
3. Under 3 years old kids which is low vital capacity, could not recommended to use it.
4. Recommended storage conditions: Unopened products in a temperature range of -20°C to 30°C with low relative humidity. Store at 80%.

PRODUCT MANUAL

Product name: Non-medical protective mask

Product material: meltblown filter material, non-woven fabric

Manufacturer: Dongguan Sunrimoon Medical & technology Co., Ltd.

Address: Shuibian Industrial Park, Hengli Town, Dongguan City

Production year: In 2020

Validity: 3 years (unopened)

Place of Origin: Dongguan

Note: Manufacturer's information for reference